## Ultra Deluxe Inflate Blood Pressure Monitor with Heart Sense®

Made in Taiwan

### Walgreens c/o SAI

105 CHALLENGER ROAD, 6FL  
RIDGEFIELD PARK, NJ 07660

### Toll Free Customer Service

1.800.261.0130

- Large Three Row Digital Display
- 3 Zones of 40 Memory - 120 Memory Total
- Time and Date
- Pulse Display
- Memory Average of last 3 Measurements
- AC Capability and Adapter

Distributor: Walgreen Co.  
Deerfield, IL 60015

## In This Manual

Important Warnings/Medical Disclaimer	1
Healthy Living®	2-3
Commonly Asked Questions	4-7
Intended Use	7
Accuracy	8
Measurement Method	8
Special Conditions	9
Special Features	9
Main Unit and Accessories	10
Digital Display Symbols	11
Battery Installation	11
AC Capability and Adapter	12
Selecting Time and Date	13
Important Reminders	14-15
Applying the Cuff	16
Taking a Blood Pressure Measurement	17-18
Storing Measurements	19
Recalling Stored Measurements	19
Reset Function	20
Taking Care of Your Monitor	21
Troubleshooting	22-23
Limited Lifetime Warranty	24-25
Specifications	26
Blood Pressure Diary	27-29

## Important Warnings/Medical Disclaimer

Before using your blood pressure monitor, you should read and understand all instructions and follow all warnings.

Information in this manual is provided for informational purposes only. This manual and product are not meant to be a substitute for the advice provided by your own physician or other medical professional. You should not use the information contained herein or this product for diagnosing or treating a health problem or disease, or prescribing any medication. If you have or suspect that you have a medical problem, promptly contact your healthcare provider.

## Healthy Living®

This blood pressure monitor is the first step to living a healthier life. But monitoring your blood pressure at home is just the beginning, it's also important to:

**EXERCISE REGULARLY** - Start out by taking a short walk each day. Before you know it, you'll feel energized. That's because along with being an easy way to shed a few pounds, walking also helps bones and hearts to become stronger.

**EAT RIGHT** - Remember, you are what you eat! So when you reach for a snack, try grabbing a piece of fruit or a vegetable. When cooking meals at home try to use fats and oils sparingly. Sometimes a healthy diet is the best medicine of all! It's also a good idea to monitor your sodium intake because about one-third of individuals who have high blood pressure are also sodium sensitive.

## Healthy Living®

**BE HAPPY!** - A positive attitude is a big part of a healthy body. Try not to get too stressed out over things. Daily relaxation techniques such as gentle yoga or even quiet time by yourself can do a lot of good for the mind and body. Stay focused on the good things in life. Begin by congratulating yourself for that short walk you just took or the smart food choice you made. It's your first step to a healthier life!

**KEEP AN EYE ON YOUR WEIGHT!** - Surprisingly, losing even a small amount of weight can often times help control high blood pressure. Talk with your doctor to find a "healthy" weight for you and strive to come within a range of it.

**Remember** to consult your physician prior to beginning exercise, vitamin/mineral/dietary changes or programs and blood pressure monitoring.

## Commonly Asked Questions

### ***Why Should I Monitor My Blood Pressure At Home?***

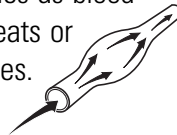
By monitoring at home, you can manage your health between doctor visits. Home monitoring also provides you with a record of blood pressure measurements over time to give to your doctor - a valuable tool in monitoring hypertension.

Home monitoring may also help alleviate *White Coat Hypertension*. This term refers to the medical phenomena which causes many individuals' blood pressure to be higher when taken in a medical office setting. That's because many times the stress of being in the doctor's office may cause blood pressure to rise as much as 20-50 points higher than normal. When monitoring at home, you can get a truer indication of your blood pressure in a normal, relaxed state.

### ***What Is Blood Pressure?***

The commonly used term "blood pressure" refers to the force or *pressure* that is exerted on the body's arteries as blood flows through them. Each time the heart beats or *contracts* it produces pressure in the arteries.

This blood pressure is what moves blood through the body, supplying oxygen and nutrients to every organ.


## Commonly Asked Questions


When the heart contracts it sends blood out into the body. This force of blood within the arteries is called **systolic** pressure.

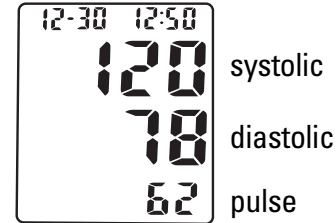The pressure that remains as the heart relaxes between beats is called **diastolic** pressure.

### ***What Is Considered Normal Blood Pressure?***

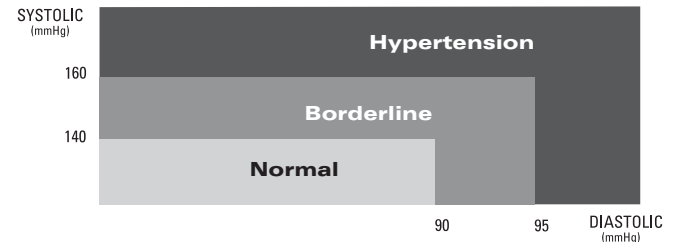The well-respected World Health Organization (WHO) has set forth the following guidelines regarding high (commonly referred to as hypertension), low and borderline blood pressure.

## Commonly Asked Questions

### ***Why Does My Blood Pressure Fluctuate?***

Because blood pressure changes with every beat of the heart it is in constant fluctuation throughout any given day. In addition to these natural changes there are other factors that may affect blood pressure such as:

- Temperature (too hot or cold)
- Humidity
- Restlessness
- Lack of sleep
- Physical exercise
- Constipation
- Stress
- Fear
- Anger
- Anxiety
- Food and beverage consumption
- Individual physical conditions

In addition to these factors you should talk with your physician to see what may be affecting you.

### ***How Do I Know I Have High Blood Pressure?***

It's nearly impossible to know if your blood pressure is elevated without having it checked. People have the common misconception that one can "feel" their blood pressure rise. It is because of this mystery that hypertension has been named "the silent killer." Don't let it sneak up on you! Start monitoring your blood pressure every day with your new Walgreens monitor.

## Commonly Asked Questions

### ***Why Is It Important To Control My Blood Pressure?***

Left uncontrolled, hypertension can place an incredible amount of stress on your heart and arteries, both of which become forced to work harder to keep blood flowing in a healthy manner throughout the body. Over time, this stress may result in health problems including over-enlargement of the heart, loss of elasticity of the arteries, heart disease, stroke, and even death. Making some simple lifestyle changes now may help you to stay healthy in the future.

## Intended Use

Walgreens blood pressure monitors are intended for home use. They are noninvasive, meaning no part of the monitor enters your body, and provide systolic, diastolic and pulse measurements. Walgreens monitors are recommended for use by people over the age of 18 and are not suitable for clinical use.

## Accuracy

Walgreens monitors have been clinically tested against a scientific device called a *sphygmomanometer*, considered the gold standard in blood pressure measurement. All Walgreens monitors have performed equivalent to measurements taken with this scientific device and are within the accuracy limits prescribed by the American National Standard for Electronic or Automated Sphygmomanometers.

## Measurement Method

Walgreens monitors measure blood pressure and pulse by using what's called an *oscillometric* method, meaning they measure the fluctuations in pressure. This is done by first wrapping a fitted cuff around the upper arm. The monitor then automatically fills the cuff with air creating pressure around the arteries inside of the arm. Mounted inside of the cuff is a special gauge that is capable of sensing the small *oscillations* (fluctuations) in pressure. These fluctuations are produced as a result of the pressure the cuff has placed around the arm and are, in a basic sense, the arteries pulsing (getting larger and smaller) with each heart beat. The monitor then measures how high and how quickly these pulses occur and converts that information into a digital value. A special valve, called an *electrode*, deflates the monitor automatically when the measurement is complete.

## Special Conditions

All Walgreens blood pressure monitors are safe and accurate to use with arrhythmias such as atrial or ventricular premature beats and atrial fibrillation.

## Special Features

This Walgreens monitor with Heart Sense® technology senses your blood pressure before the measurement is complete - the monitor then adjusts the air pressure amount according to your individual level. The results are quick, comfortable measurements each time you check your blood pressure.

### **Large Three Row Display with Backlight**

A large LCD with a backlight illuminates the time, date and your blood pressure - systolic over diastolic - to make measuring your blood pressure easy to read.

### **Memory Feature**

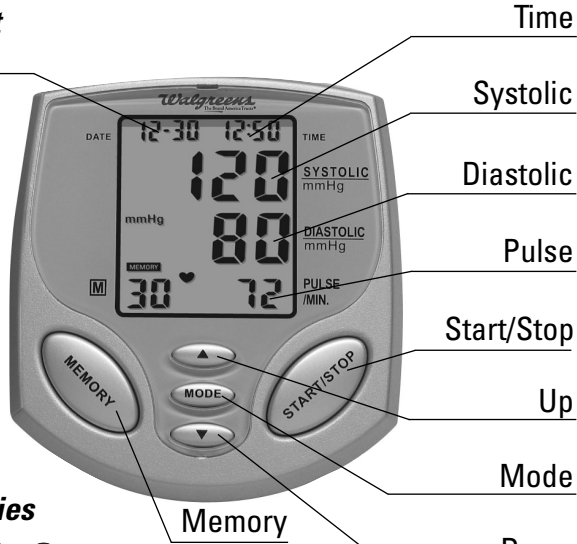
Your Walgreens Ultra Deluxe Inflate monitor features a 3 party memory capability or 3 "zones" - each zone stores 40 measurements for a total storage capacity of 120 measurements. This unique feature allows 3 users to use and store their blood pressure readings separately all on one monitor!

### **Time and Date**


This monitor automatically keeps track of the time and date and remains on when the monitor is not in use for easy reference.

## Main Unit and Accessories


Main Unit  
Date

### Accessories

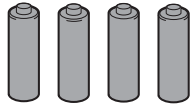
4 "AA" Alkaline Batteries

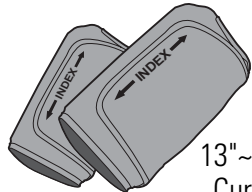
9"~13" Regular  
Curved Cuff

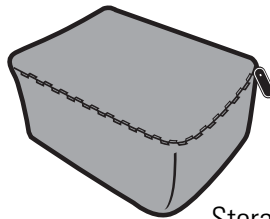13"~17" Large  
Curved Cuff


Storage Pouch

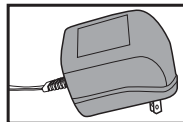AC Adapter

## Digital Display Symbols

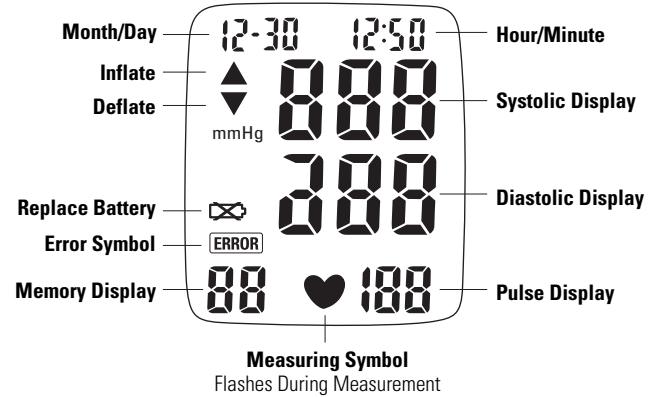
## Battery Installation

When the display shows  $\times$ , it's time to replace the batteries. This monitor uses 4 "AA" alkaline batteries.

Insert the batteries into the monitor making certain to align the polarities correctly.

Close the battery cover securely. It will snap into place when it is fully closed.

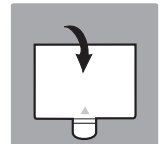\* It's a good idea to remove the batteries if you know you will not be using your batteries for an extended amount of time.


## AC Capability and Adapter

This Walgreens blood pressure monitor is equipped with AC capability should you choose to use electric power instead of batteries to operate your unit.

Use only the authorized AC adapter included with your monitor. Use of any other adapter may cause damage to the monitor and will void the user warranty. For warranty or replacement information refer to pages 24-25.
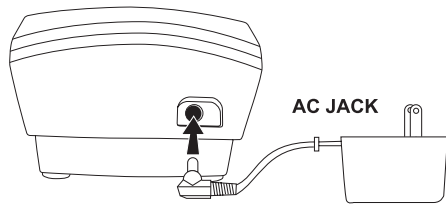
### AC Adapter Output: 6V DC 1A

\*It is suggested that you remove the batteries from the compartment if you are going to use an AC adapter for an extended amount of time.

## Selecting Time and Date

Your Walgreens monitor will keep track of the time, month, and date unless the batteries have run down or are removed from the monitor.

- Press the MODE button on the unit. The month will begin to flash. Press the ▲ (Up) or ▼ (Down) button until the correct month appears on the display.
- Press the MODE button once again to store and now the day will begin to flash. Press the ▲ (Up) or ▼ (Down) button until the correct day appears on the display.
- Press the MODE button to store and the hour will begin to flash. Follow the same process as setting the month and day to set the hour and minute.
- Once you have selected the minute, press the MODE button. The unit will now display the time and date.

## Important Reminders

Here are some important reminders:

### **Before Taking a Measurement**

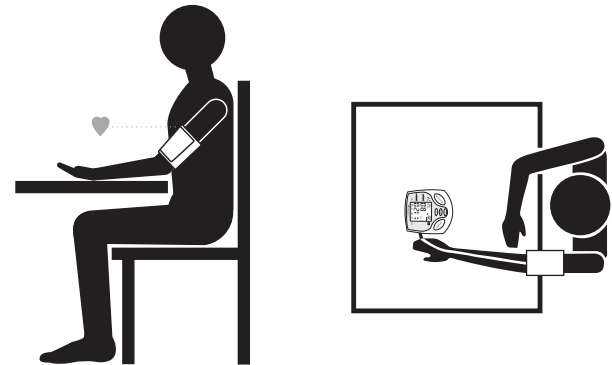
- ✓ Sit quietly for about 10 minutes. This will allow your body to return to its normal, resting state.
- ✓ Take your blood pressure in a comfortable environment - not too cold, not too hot - as temperature may affect your measurement.
- ✓ If you've just consumed beverages containing caffeine such as coffee, non-herbal tea, or cola wait about 30-45 minutes. Also, don't smoke just before taking a measurement.
- ✓ Insert the cuff plug into the monitor before taking a measurement.

## Important Reminders

### **While Taking a Measurement**

- ✓ Sit with your legs uncrossed and your feet flat on the floor.
- ✓ For consistency, it's a good idea to measure your blood pressure using the same arm and at about the same time each day. If possible, we suggest using the left arm.
- ✓ Relax. Sit still and quietly while measuring. Talking or moving may elevate measurements. Do not touch the cuff or monitor during the measurement.
- ✓ When taking multiple measurement right after each other, make to wait at least 8-10 minutes in between. Waiting allows your blood vessels to return to their normal state.

## Applying the Cuff

1. If your arm measures 9"-13" around, use the regular cuff included with your monitor. If your arm measures 13"-17" around, use the large cuff included with your monitor.

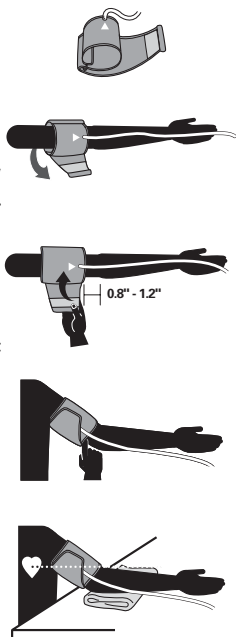
2. Before taking a measurement, read the Important Reminders on pages 14-15. Make certain the cuff plug is inserted securely in the monitor. Remove any constricting or thick clothing from your upper arm.

3. Turn your palm upwards. The cuff should be positioned on your bicep and the red arterial marker should be in the center of the inside your arm over the brachial artery. The cuff tube should also run down the center of your arm. Instructions are also printed directly on the cuff for reference.

4. Place the curved cuff on your bicep and secure the cuff firmly around your arm using the Velcro.

5. The velcro tab should fall within the "OK" range printed on the cuff for proper use. If your arm is larger than 13" around the cuff does not fall within the "OK" range, use the large cuff enclosed in the package.


6. Relax your arm and place your elbow on the table.

## Taking a Blood Pressure Measurement

1. Once the cuff is properly applied and you are seated with your feet flat on the floor you are ready to take a measurement.

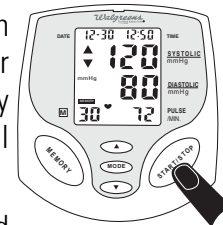
2. Press the ▲ (Up) or ▼ (Down) button and you will see a number appear: 1, 2, or 3. These are each of the 3 party memory zones explained on page 9 (Special Features).

\*If you select zone No. 1 your blood pressure will be recorded as one of the 40 readings available for that zone. The same is true for zones 2 and 3. When taking measurements, be sure and select the correct zone number so that the measurement is stored in the appropriate zone.

3. Press the START/STOP button. The monitor will begin to inflate to the level that's right for you. An upwards facing arrow on the digital display indicates the monitor is inflating. Remain still and do not talk during the measurement process.

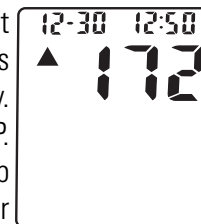
4. You will feel the cuff get tighter around your arm - this is normal. It should not inflate to the point of discomfort. If this occurs, the cuff may be secured too tightly. To stop the inflation, press START/STOP. The cuff will deflate automatically. Rewrap the cuff allowing more room between your arm and the cuff.

Inflation


## Taking a Blood Pressure Measurement

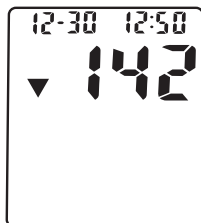5. Inflation will stop automatically to begin the measurement process. You will see a downward facing arrow and the numbers decreasing as the cuff is deflating. You will also see a flashing heart to indicate the measurement is taking place.

Deflation

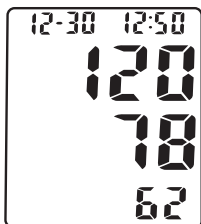Note: Occasionally, to get an appropriate measurement, the monitor may reinflate the cuff to 30mmHg higher than the previous inflation and begin the measurement process again.

6. When your measurement is complete the monitor will deflate automatically, releasing the air pressure in the cuff.

7. The entire measurement process should take less than a minute. Your blood pressure and pulse readings are displayed and automatically stored in the memory.

\* The monitor will automatically power off after about one minute.

**NOTE:** Blood pressure constantly changes during the course of a day. Occasionally, high or low readings do not necessarily mean your monitor is malfunctioning or that you are ill. It is important to speak with your doctor to learn what your normal blood pressure is if you experience abnormal measurements.

## Storing Measurements

When you take a measurement the monitor automatically stores it in memory space number one. As you continue to take measurements the monitor will push the previous measurement back one space. In other words, the value in space one will always be the most recent blood pressure measurement taken.

Your monitor features a 3 party memory capability or 3 "zones" - each zone stores 40 measurements for a total memory capacity of 120 readings. This unique feature allows 3 users to use and store their blood pressure measurements separately all on one monitor. This feature is also useful for one user to track their blood pressure throughout the day: morning, afternoon, and evening.

## Recalling Stored Measurements

1. Press the ▲ (Up) or ▼ (Down) button to select a user zone. You will see a number appear: 1, 2, or 3. (Fig. 1)
2. Press the MEMORY button and the average of the last 3 readings will be displayed for that zone. For example, if you select zone No. 1 and you press MEMORY, the average of the last 3 measurements stored in zone No. 1 will appear on the display.
3. Press ▼ (Down) to view the next stored measurement; press ▲ (Up) to go back to the previous measurement.
4. To go back to the main screen for "Recalling Stored Measurements" and to choose another zone, press ▲ (Up) to scroll through the measurements until you reach the most recent measurement and press once more. You can also press START/STOP to return to the main screen.

## Recalling Stored Measurements

5. Follow steps 1-3 once again to review measurements in another zone.

\*Fig.1

\*Fig.2

## Reset Function

Your Walgreens monitor will not lose the data stored in the memory even when you change the batteries. To erase all of the data in memory zone, refer to the instructions below.

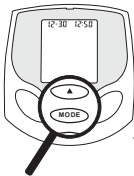
Note: Single measurements cannot be erased.

Warning: Resetting the memory will permanently erase all of the stored data in the memory zone that you select.

- Press ▲ (Up) ▼ (Down) to choose a zone: 1, 2, or 3.
- Press the MEMORY button. (Fig. 3)
- Press the MODE and ▲ (Up) button at the same time to clear all of the stored data. (Fig. 4)

\*Fig.3

\*Fig.4

## Taking Care of Your Monitor

It's important to treat your monitor with care. Here are some basic tips:

- ✓ Keep the monitor out of direct sunlight, high humidity and extreme temperature changes. 
- ✓ Try not to drop it on the floor.
- ✓ Do not take the monitor or the cuff apart. None of the monitor's parts are fixable at home. If your monitor needs repair, see details in the warranty section of this manual.
- ✓ Keep the monitor free of dust and other materials. Avoid getting any part of the monitor or cuff extremely wet. If the monitor needs cleaning it's best to wipe it with a cloth slightly dampened with water. We recommend that you clean your monitor about once a month.
- ✓ Do not harshly fold or bend the cuff as  this may damage it on the inside.
- ✓ Do not inflate the monitor's cuff if it is not wrapped around your arm.
- ✓ Always remove the cuff tube from the monitor with care.
- ✓ For your safety, do not wrap the cuff around any other body part other than your arm.

## Troubleshooting

**Error/Symbol    What Does It Mean?    How To Fix It**

<p><b>ERROR</b></p>	<p>Monitor may have been inflated without the cuff around the arm <i>OR</i> Cuff may not be positioned correctly on the arm.</p>	<p>Read page 16 (Applying the Cuff).</p>
	<p>The cuff tube may be folded <i>OR</i> You may have moved while the cuff was inflating.</p>	<p>Read pages 14 &amp; 15 (Important Reminders).</p>
	<p>You may have moved or spoken during the measurement process.</p>	<p>Remember that it is important not to move and speak as it may elevate measurements.</p>
	<p>You may have been startled by a noise or vibration while measuring.</p>	<p>Make sure you are in a quiet and comfortable place when taking a measurement.</p>

## Troubleshooting

**Error/Symbol    What Does It Mean?    How To Fix It**

<p><b>ERROR</b></p>	<p>You have been ignoring the flashing replace battery symbol (  ).</p>	<p>Replace with four new "AA" alkaline batteries. Make sure the polarities are positioned correctly in the battery compartment before continuing to measure your blood pressure.</p>
<p></p>	<p>The batteries have run down.</p>	<p>Replace with 4 "AA" batteries so you may continue to measure your blood pressure without any interruptions.</p>
<p>Blood pressure value seems too high or too low</p>	<p>Cuff may not be aligned correctly.</p>	<p>Read page 16 (Applying the Cuff).</p>
<p>Blood pressure value seems too high or too low</p>	<p>Circuitry locked.</p>	<p>Remove then reinsert the batteries and then proceed to take measurement again.</p>
<p>Unable to take a measurement after receiving an error symbol.</p>	<p>The cuff may not be positioned correctly.</p>	<p>Read page 16 (Applying the Cuff).</p>

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