# QUICK REFERENCE GUIDE

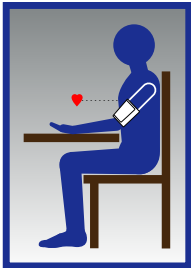
**Walgreens**  
The Brand America Trusts®  
BD-7181W

## 1 BEFORE TAKING A MEASUREMENT

1 Sit quietly in a comfortable environment for about 10 minutes. Try to refrain from talking and moving.

2 Wait 30-45 minutes before taking a measurement if you've just consumed any caffeinated beverages or smoked cigarettes /cigars.

3 For consistency, try to take your blood pressure using the same arm, we suggest the left.

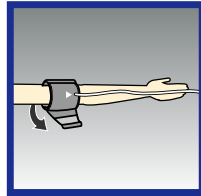
4 Sit with your legs uncrossed and your feet flat on the floor. Do not touch the cuff or monitor while measuring.

5 Wait at least 10 minutes in between measurements.

6 Note: If your arm measures more than 13", please use Walgreens Large Cuff (Model #BC-200W)

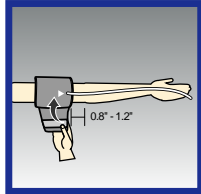
## 2 CUFF PLACEMENT

1 Make sure you are not wearing a thick coat or sweater. If you're wearing a long sleeved shirt,

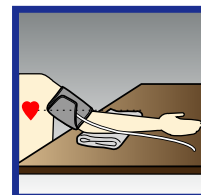
roll up the sleeve so it's not constricting circulation to your arm.

2 Wrap the cuff around your arm making sure that it's not too tight.

For detailed instructions on how to apply the cuff, refer to p. 16 in the manual.

3 Place your arm flat on a table with your palm facing upwards.

# QUICK REFERENCE GUIDE

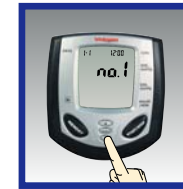
**Walgreens**  
The Brand America Trusts®  
BD-7181W

## 3 TAKING A MEASUREMENT

1 Once the cuff is properly applied and you are seated with your feet flat on the floor you are ready to take a measurement.

2 Press the ▲ (Up) or ▼ (Down) button to select a memory storage location: 1, 2, or 3. These are each of the 3 party memory zones explained on p. 9 of the instruction manual (Special Features).

3 Press the START/STOP button. The monitor will begin to inflate to the level that's right for you. An upwards facing arrow on the digital display indicates the monitor is inflating. Remain still and do not talk during the measurement process.

4 As the cuff inflates you will feel it tighten and numbers will appear on the digital display. This is the normal process of measurement.

5 Once measurement is complete the monitor will automatically deflate and your results will be displayed on the digital panel.

6 The monitor will automatically power off after about 1 minute.

Walgreens c/o SAI  
105 CHALLENGER ROAD, 6FL  
RIDGFIELD PARK, NJ 07660

Toll Free Customer Service  
1.800.261.0130

Heart Sense® and Healthy Living® are registered trademarks used under license by Samsung America, Inc.

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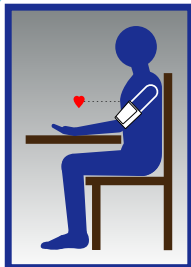
**Walgreens**  
The Brand America Trusts®

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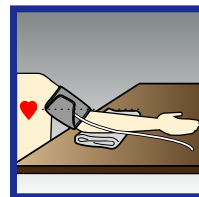
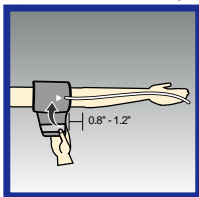
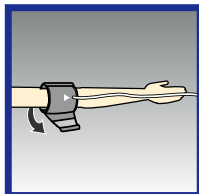
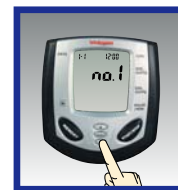
Walgreens c/o SAI  
105 CHALLENGER ROAD, 6FL  
RIDGEFIELD PARK, NJ 07660

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La marca registrada de Sentidos del Corazon® y una Mejor Salud® estan bajo la licencia de Samsung America, Inc.